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CoralPress PUTTY

Hydrophilic base impression material for the putty/wash technique

CoralPress PUTTY is used as a hydrophilic base impression material for the putty/wash impression technique in conjunction with **CoralPress LIGHT**. **CoralPress PUTTY** is a kneadable precision impression material based on addition curing vinyl silicone. **CoralPress PUTTY** is easy to mix and not sticky. It meets the requirement **ISO 4823** (2000), **type 0** (putty consistency).

Application

Perforated or non-perforated metal or plastic trays may be used with **CoralPress PUTTY**. In order to improve adhesion of **CoralPress PUTTY** to the tray, a silicone adhesive should be applied as a thin even film. Allow to dry **for 3 minutes** before loading the impression tray.

Use **equal parts** (volume or weight) of **base paste** and **catalyst paste**. Take the base paste with the dark measuring spoon and the catalyst paste with the white measuring spoon. Knead until the colour is homogeneous (at least **30 seconds**). Load the ready kneaded material into the prepared tray. Insert the loaded tray in the mouth not later than **1:30 minutes** (working time at 23°C or 74°F) after start of mix. Maintain position for at least **3 minutes**.

The impression should be stored at normal room temperature (max. 25 °C or 78 °F).

Casting

The impression can be poured **30 minutes** after removal from the mouth. No other time limitation does exist.

Electroplating

All impressions can be copper- or silverplated.

Desinfection

Impressions can be disinfected with standard disinfection solutions.

Storage

Do not store above 25 °C (78 °F). Do not use after expiry date

Additional notes:

Do not interchange lids of pots. Do not interchange measuring spoons.

Do not use with standard (condensation reaction type) silicone impression materials.

Do not use Latex gloves, they may interfere with the setting reaction of the impression material.